vyvo™

Y **Icon**

The most advanced
health watch.

QUICK START GUIDE

EN

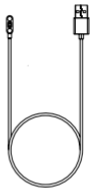
1. PACKING LIST

The box contains the following items:

ICON

USB Charging Cable

Instructions

2. HOW TO OPERATE

2.1. Charging

Properly attach the magnetic charging base to the back of the watch, then plug the charging cable into a USB port on a computer, charging dock or power bank for charging.

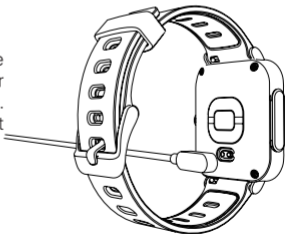
Input voltage: 5V DC.

Input current: <0.3A.

Charging time: About 2 hours.

Note:

1. It is recommended to use the universal 5V/1A charger with 3C logo on the market.
2. Do not use a fast charging charger.

2.2. APP Download

The VYVO Smart App is available for iOS in the Apple App Store and for Android in the Google Play Store. Please search for “VYVO Smart” to download and install the app.

2.3. Compatibility & Support

Support Android 4.4 or above, iOS 9.0 or above.

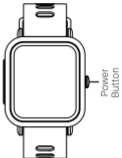
2.4. Connection

After downloading, open the app, login or register an account, fill in your essential personal details (height, weight, date of birth), and then complete the connection according to the operation instructions on the app.

Note:

1. In order to connect the phone to the watch successfully, you need to turn on the Bluetooth on your phone and connect to your device through the app.
2. The new version of the APP on the Android phone requires you to turn on the “Location Information” on your phone to be connected.
3. The first time you connect to the APP, the date & time on the phone will be synchronized to the watch, and previous data for steps, calories & distance on the watch will be cleared.

2.5. Device Overview

	SHORT PRESS	LONG PRESS	
Power Button	<ol style="list-style-type: none"> 1. Turn on the watch's screen 2. Turn off the watch's screen 3. Pause/continue the exercise 4. Enter the sub-interface 	<ol style="list-style-type: none"> 1.Power on 2.Power off 	

Operation	Condition
Swipe left/right	<ol style="list-style-type: none"> 1. Swipe left to view all the watch's features. 2. Swipe right to return to the previous menu.
Swipe up/down	<ol style="list-style-type: none"> 1. Swipe up to Switch options. 2. Swipe down from the home screen to view Steps, Standing time, Calories, Air Quality Index, Pollutant (App sync required).
Tap the screen	Enter the sub-interface.
Press & hold the screen	From the watch home screen, press and hold to change the watch face.

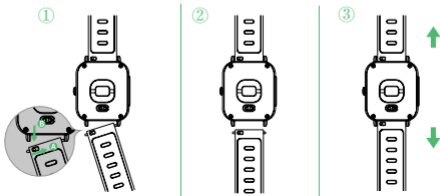
2.6. Wearing the Device

1. For optimized tracking of the measured values, we recommend wearing the device with one finger's width below your wrist bone.
2. Please make sure the device is fairly snug against your skin and does not slide up or down your wrist during exercise.

2.7. Replace the wristband

1. Remove the wristband from the watch by sliding the snap lock on the wristband.
2. Align the new wristband with the watch and buckle it in.
3. Pull the wristband lightly to make sure it is buckled in the watch.

3. INTRODUCTION FOR IMPORTANT FEATURES

3.1. Heart rate monitoring

ICON contains a PPG heart rate sensor and combines with an HR algorithm; it is capable of accurately monitor your heart rate while wearing it.

Note: the light signal transmission might be blocked if your skin is too dry or too dark or excess hair, which may lead it to a measurement failure.

3.2. Sleep Monitoring

When you wear your watch to bed in the evening, you can check your sleep data monitored on the VYVO Smart App after you get up in the morning. The watch goes into sleep monitoring mode from 8:00 pm to 9:00 am the next day.

Note:

1. The sleep monitoring function will be stopped after you get up and move for 5-10 minutes.

The watch does not record daytime sleep data.

3.3. Alarm/Schedule

Alarm/Schedule needs to be set in the VYVO Smart App; the watch will remind you correspondingly after setting it.

3.4. 15 Sports Modes

There are 15 sports modes for your choice: Run, Cycle, Fast Walk, Climb, Basketball, Badminton, Table Tennis, Tennis, Ice-skating, Soccer, Volleyball, Yoga, Aerobics, Dance, Swimming. You can choose the one that suits your activity best.

Pause/Continue/End Activity: press the power button to pause during exercise, the paused interface is displayed, and you can tap the left square icon to choose whether to end the activity or tap the right triangle icon to resume.

View Exercise Data: slide up or down to view different data during exercise; after exercising, you can directly view the activity data on the watch. Detailed data can be viewed after pairing and synchronizing your watch with the VYVO Smart App.

Note: you can choose to turn on or off the GPS and the alert of every kilometer/mile before running, cycling, fast walking, or climbing.

4. MORE INFORMATION

4.1. Waterproofing instructions

The waterproof level of ICON is IP68 (50m).

The water resistance performance of the device is not permanently valid; it may diminish as time passes. The device can be used during washing hands, rain or swimming in shallow water, but not support for hot water showers, diving, surfing, etc.

It does not have a waterproof effect on corrosive liquids such as seawater, acidic and alkaline solutions, and chemical reagents. If you inadvertently encounter corrosive liquid, please clean with clear water and wipe it dry. The damage caused by abuse or improper use is not covered by the warranty.

The following conditions may affect the water resistance performance, and should be avoided when using:

1. The watch falls, bumps, or suffers from other impacts.
2. The watch is exposed to soap water, shower gel, detergent, perfume, lotion, oil, etc.
3. Hot and humid scenes such as hot baths and saunas.

4.2. Firmware Upgrade

When there is a new firmware version, there will be a notification in the VYVO Smart App. Go to the Device interface of the app and select the firmware upgrade.

Notes:

1. Make sure that the battery level is more than 50% before upgrading.
2. During the upgrade process, you cannot quit halfway if the progress bar moves, keep the screen of your phone bright and active, and only when the upgrade is completed can you exit the interface. Otherwise, the upgrade will fail.
3. If the upgrade fails, wait for the watch to restart automatically when the upgrade fails. Then reconnect to the app to attempt the upgrade again.

4.3. Specification parameter

Product model	ICON
Size	49 × 37 × 13.7 mm
Adjustable range of wristband	150 mm - 250 mm
Battery life	15 days (receive an average of 50 messages & 5 calls per day and raise your wrist to wake the screen up 50 times; turn on 24-hour automatic heart rate monitoring).
Battery capacity	Li-Polymer battery 240mAh
Weight	About 45g
Working condition	0-40°C
Display size and Screen resolution	1.3 inch TFT color square display 240*240 px

5. DEVICE MAINTENANCE

5.1. Device care

Notice:

1. Do not use a sharp object to clean the device.
2. Avoid using solvents, chemical cleaners, or insect repellents that might damage plastic components of the device.
3. Thoroughly rinse the device with fresh water after exposure to chlorine, salt water, sunscreen, cosmetics, alcohol, or other harsh chemicals. Prolonged exposure to these substances can damage the device.
4. Avoid extreme shock and harsh treatment since it can reduce product life.
5. Avoid pressing the button underwater.
6. Do not expose to extreme temperatures that are too cold or too hot for a long time, which can cause permanent damage.
7. After each training session or exercise, please rinse the watch with clean water. Sweat may interfere and cause inaccuracies.

5.2. Cleaning the Device

Note:

Even the fainter sweat, or moisture may cause corrosion on the charging terminal when charging the device, which will also hinder the transmission of data and affect charging.

1. Gently wipe the device by using a cloth with a mild neutral detergent.
2. Wait for dry after cleaning.

6. IMPORTANT SAFETY AND PRODUCT INFORMATION

- If you have a pacemaker or other internal electronic device, consult your physician before using a heart rate monitor.
- The inside optical wrist heart rate monitor will occasionally emit green light and flashes. Consult your physician if you have epilepsy or are sensitive to flashes of light.
- Always consult your physician before beginning or modifying any exercise program.
- The device, accessories, heart rate monitor, and related data are intended to be used only for exercise monitoring, not medical purposes.
- The heart rate readings are for reference only, and no liability is accepted for the consequences of any wrong interpretation.
- Although the optical wrist heart rate monitor technology typically provides an accurate data for user's heart rate, there are inherent limitations with the technology that may cause some of the heart rate readings to be inaccurate under certain circumstances, including the user's physical characteristics, installation of the device and type & intensity of activity.

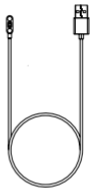
- The activity trackers rely on sensors to track your movement and other metrics. The data and information provided by the device are intended to assess your activity and metrics closely, but may not be completely accurate, including the data for step, sleep, distance, heart rate, and calories.
- Do not immerse or expose batteries to water or other liquids.
- Do not expose the device to a heat source or in a high-temperature location, for example, in an unattended vehicle in the sun. To prevent the possibility of damage, remove the device from the vehicle or store it out of direct sunlight, such as the glove box.
- When storing the device for an extended period of time, please store within the temperature ranges specified in the printed manual, which are included in the product packaging.
- Do not use a power and/or USB cable that is not approved or supplied by the manufacturer.

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